NAF Naturalint Naturalint

Veterinary approved

Hoof Poultice

3 Dressings

DIRECTIONS FOR USE

NAF NaturalintX Hoof Poultice has been designed to fit your horse's hoof, reducing preparation time and enabling easy application. The NaturalintX Hoof Poultice can be applied in three different ways:

Hot Hoof Poultice:

Place the hoof poultice in boiled water, that has been allowed to cool to hand hot temperature. Once saturated, remove the dressing and gently squeeze out excess water. Place the poultice on the affected area with the plastic backing away from the

with the plastic backing away from the skin. Hold in place with a NaturalintX Wrap.

Cold Hoof Poultice:

Soak the hoof poultice in cold boiled water before applying to the affected area. The hoof poultice can also be chilled further, by placing it in its outer plastic bag in a refrigerator.

Dry Hoof Poultice:

Simply apply the hoof poultice directly to the affected hoof. Ensure the hoof poultice is applied with the plastic away from the skin before holding in place with the NaturalintX Wrap.

CALITION:

Clean the area thoroughly before the application of NaturalintX Hoof Poultice.

Clean hands thoroughly before and after application.

Do not apply the NaturalintX Wrap or bandage too tightly, as circulation can be severely restricted.

NAF NaturalintX Veterinary approved Hoof Poultice should be renewed at least every 12 hours and should not be used in conjunction with any medical preparations unless advised otherwise by a veterinary surgeon.

WARNING Prior to applying the hoof poultice to your horse, ensure that the temperature is suitable on your own skin. Both extremes of hot and cold can be dangerous to your horse.

For further information on using this product contact: NAF, Wonastow Rd, Ind Est West, Monmouth, NP25 5JA call our Freephone Advice Line: 0800 373 106

or email info@naf-uk.com

Active ingredients:

Each Hoof Poultice contains: Tragacanth 1.12g, Boric Acid 0.39g.

TO AVOID DANGER OF SUFFOCATION, KEEP PACKAGING OUT OF REACH OF BABIES AND CHILDREN. FOR ANIMAL USE ONLY. FOR EXTERNAL USE ONLY.



Batch No: See Side Product code: 148





